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Category: machinery and appliances
general designation: household electric massage chair

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Before use

- **IMPORTANT SAFETY INSTRUCTIONS** 2
- **Place massage chair** 9
- **Before use** 11
- **product structure** 15

How to use

- **Operation Instructions** 19

Overhaul and maintenance methods

- **Cleaning and maintenance** 22
- **Troubleshooting** 23
- **Product specifications** 24

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Purpose of use, effect-efficacy

Substitution of massage for home-use

- Thank you very much for purchasing Fujiiryoki massage chair.
- Carefully read this operation manual before use and only use as directed.
- Please read the safety precautions on pages 2-8 before use.
- Keep this operation manual where it will be readily available for future referral.
- Please recycle the outer packaging material where possible.

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IMPORTANT SAFETY INSTRUCTIONS

- Read these "Important Safety Instructions" before use and use this product properly
- These important safety instructions are prescribed to ensure correct usage of the product, and to prevent injury and damage to the user and other persons.
- Followings are the display and contents.
- Please keep to the regulations as these are important instructions for your safety.

 DANGER	Improper usage may result in serious injury or death.
 WARNING	Improper usage may result in injury or damage to property.*

*Property refers to houses, household belongings, domestic animals and pets.

	<p>⊘ Prohibited actions The prohibited action is illustrated inside the illustration (for example, the illustration on the left refers to "Prohibiting disassembling")</p>
	<p>● Directives, actions which are compulsory The prohibited action is illustrated inside the illustration (for example, the illustration on the left refers to "unplugging the power plug")</p>

- Always keep this operation manual where it will be readily available for referral.
- Attention: When disposing of this product and/or its accessories, do so in accordance with local laws and regulations. For details, please refer to your local authority.

DANGER

	<p>The Because this equipment has a built-in heater and the surface becomes hot, person who are not sensitive to heat should pay attention to use. Pay full attention when it falls under the following.</p> <ul style="list-style-type: none"> •elderly, ailing person, those who cannot operate by themselves •persons who took medicine that cause drowsiness(sleeping pills, cold medicine) •persons with heavy drinking, heavy tiredness •persons with hematogenous disorder (diabetes, etc.) •persons who receive a note from the doctor •people with weak temperature sense. <p>There is a risk of low temperature burns, dehydration.</p>
	<p>The following persons must consult a medical physician before using this product.</p> <ul style="list-style-type: none"> •Persons using a medical electronic device embedded in the body such as a pacemaker susceptible to the electromagnetic disturbances. •persons with malignant tumors. •persons with heart diseases. •persons who are pregnancy or those immediately after childbirth. •persons who suffer from paresthesia related to peripheral circulatory disturbance due to diabetes etc. •persons with wounds on their skin. •persons in need of repose. •persons with Temperature sensory loss. •persons with body temperatures exceeding 38°C. (when strong acute inflammation symptoms, malaise, hills, blood pressure fluctuations are observed, when debilitated) •persons with herniated discs. •a person with tendovaginitis. •persons who suffer from osteoporosis, spinal column fractures, or acute pains. •persons with abnormalities in the spine, or those whose spines are bent. •persons who feel an abnormality on the body not prescribed above, persons in treatment by a doctor.

DANGER

If the chair stops moving or any abnormalities occur, unplug the power plug and contact the point of purchase for an inspection/repairs.
Electrification, short circuits and fire may occur.

Take care of the movements of the knead balls when massaging around neck. Avoid excessive massaging or massaging the front of the neck.
Accidents or injuries may occur.

When reclining or lowering the footrest, always check underneath and surroundings the footrest for people, pets, and other items.
Accidents or injuries may occur.

When reclining or lowering the footrest, be careful not to get hands, fingers, feet, head caught underneath the footrest or backrest.
●underneath the chair itself ●backrest, seat, clearance of armrest ●clearance of footrest, seat, armrest ●underneath the footrest, the back of footrest ●clearance between arm and armrest ●behind the backrest
Accidents or injuries may occur.

Before use, check underneath the back pad to make sure that there are no rips or tears in the cover cloth of the seat back. (should there be any rips or tears, no matter how small, immediately discontinue use, disconnect the power cord, and ask for repairs to be conducted).
Continued use with rips or tears in the cloth of the seat back may cause electric shocks or injuries.

Always unplug this appliance from the electrical outlet immediately after using and before cleaning.

Not long term use of heater and same spot on the body
If heat is used on the same spot for a long time, low temperature burns may occur even if the temperature is comparatively low in 40-60°C.

— About low temperature burn —
●low temperature burn effect the deep part of the skin and show red specks or blisters.
●In case red specks or blisters are found, consult a medical physician immediately.

The following persons are strongly recommended not to use this product. Certain prohibited from receiving a massage by a medical physician.
(for example those suffering from thrombosis, severe aneurysm, acute varicose veins, various kinds of dermatitis and skin infections (including inflammations of subcutaneous tissue), etc.)

Do not remove, rip or cut the cover of the chair(backrest, footrest) on purpose.
Never use the chair with the cover removed or torn as clothes and hair may get caught in the knead balls.

Do not use the chair other than on the designated voltage. Fire or electrification may occur.

Do not use the chair when the power cord or its plug is damaged, or when the receptacle is loose. Contact your point of purchase for repairs if the power cord or power plug are damaged in anyway. Continued usage may result in electric shocks, shortage, or even fire.

Surely plug the power plug to the root and keep away from the dust. Doing so may cause damage, fire and electrification.

Do not allow unattended children or persons unable to declare their own intentions to use this chair. Do not allow infants near the chair. Electrification or injuries may occur.

Do not allow children to play or get on the chair. Accidents or injuries may occur.

Do not connect or disconnect the power plug with wet hands. Electrification may occur.

IMPORTANT SAFETY INSTRUCTIONS

DANGER

	Never use pins or other metallic fasteners with this appliance.
	Carefully examine the covering before each use. Discard the appliance if the covering shows any sign of deterioration, such as checking, blistering, or cracking.
	Do not use or store in the bathrooms or places of high humidity or moisture. Electrification, fire, accidents, and moulding may occur.
	Never remodel or decompose/repair by yourself. Fire or injury due to abnormal action may occur.

WARNING

	Usage of this product should be limited to 30 minutes for each use. Continual use on the same part of the body should be limited to 5 minutes. Prolonged use may damage muscles and nerves.(Only use for 30 minutes per day.)
	The following persons should consult a medical physician before using this product. <ul style="list-style-type: none"> ●persons whose muscles have become weak due to increasing age, and those who are of slim build. ●persons suffering from lower back pain caused by bones or internal organs. ●persons easily bruised, susceptible to sprains. ●persons who suffer from intense motion sickness. ●persons who have undergone internal organ or heart operations.
	If any rashes, reddening of the skin, itches or similar symptoms appear after using this product, immediately stop using and consult a physician. Accident and ill health may occur.
	An appliance should never be left unattended when plugged in. Unplug from outlet when not in use, and before putting on or taking off parts. To disconnect, turn all controls to the off position, then remove plug from outlet. This may result in accidents due to children's mischief.
	Use the chair on a level surface. Do not use outdoors. Damage or accidents may occur.
	Make sure operation buttons and timers are operating normally before use. Accidents or injury may occur.
	Read the operation manual once again and confirm the product is operating normally before using the chair after a prolonged period of non-use. Accidents or injury may occur.
	Confirm no items get caught in the chair (backrest, footrest, armrest) before use. Never drop or insert any object into any opening. Accidents or injury may occur.
	Do not place feet or hands on the footrest when the footrest is retracting. Accidents or injury may occur.
	When any physical abnormalities appear or are felt during operation, stop using the product immediately and consult a physician.
	Sit down slowly after confirming the knead balls are located at the storage position. Accidents or injury may occur.
	Use weak stimulation massage when you start to use. Injury may occur.

WARNING

	Make sure the ground terminal is installed. Damages or electrification may occur when ground terminal is not installed.
	Turn the power switch OFF and unplug the power cord immediately in times of power shortages. When power is restored accidents may occur.
	Unplug the power plug by holding the actual power plug and not the cord. This may cause electrification, short circuits, or fire.
	Unplug the power plug when not using the chair. This may cause a fire due to insulation degradation by dust or humidity.
	When maintaining the chair always unplug the power cord. This may cause electrification or injury.
	Never operate the appliance with the air openings blocked. Keep the air openings free of lint, hair, and the like.
	Never drop or insert any object into any opening.
	To disconnect, turn all controls to the off position, than remove plug from outlet.
	The appliance is intended for household use.
	Temperatures sufficiently high to cause burns may occur egardless of the control setting. Do not use on an infant or invalid or on a sleeping or unconscious person. Do not use on insensitve skin or on a person with poor blood circulation. Check the skin in contact with the heated area of the appliance frequently to reduce the risk of blistering.
	Do Not Crush - Avoid sharp folds.
	Do not damage the power cord or power plug. <ul style="list-style-type: none"> ●Do not hurt ●Do not damage ●Do not process ●unreasonably bend or pull. ●Do not twist ●Do not tie ●Do not place heavy items on the power cord. ●Do not get the power cord caught, especially when moving the chair. Never operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the appliance to a service center for examination and repair. Continued usage may result in electric shocks, shortage, or even fire.
	Do not use immediately after eating or consuming alcohol. Nausea, illness, accidents or injury may occur.
	Do not close to stoves or other sources of fire, nor smoke whilst using the chair. Do not use on heated carpets or other geating equipment. Do not operate under blanket or pillow. Excessive heating can occur and cause fire, Electric shock, or injury to persons. Keep the cord away from heated surfaces. This may cause fire.
	Do not stand on the seat, or get on the seat back, footrest, arm or shoulder portion of the chair. The user or chair may rollover and injury or accidents may result.
	Do not stand or sit down on upper part of the backrest, the front of the seat or on the footrest when the chair is reclining. The user or chair may rollover and injury or accidents may result.
	Do not try to sit or get up from the chair when the chair is reclining or the footrest is elevated. The user or chair may rollover and injury or accidents may result.

IMPORTANT SAFETY INSTRUCTIONS

WARNING

Do not use accessories other than those which accompany the chair. Doing so may cause product malfunction.

Do not use the chair whilst wearing earrings, necklaces, scarves, or other ornaments. This may cause injury.

Do not use the chair whilst wearing hard objects on the head, such as hair ornaments. This may cause injury.

Do not use the thigh or hip massage with hard objects in pockets. This may cause accidents or injury.

Do not use while naked. Direct massages on the naked body may damage the skin.

Do not use on the elbow, knee, head, abdomen areas. Do not get hands or feet caught in the knead balls. Injuries may occur.

Do not pull out the power cord, or turn off the power switch whilst using the chair. This may cause accidents or injury.

Do not sleep whilst using the chair. This may cause ill health or injury.

Do not move the chair while people or items are on the chair. Damage or injuries may occur.

Make sure no more than 1 person is on the chair at any one time. Doing so may result in damage to the chair or injury.

Do not wipe with benzine, paint thinner or alcohol, nor spray with any pharmaceutical sprays. This may cause fire or electrification.

Not to be used for purposes other than massage purposes. This may cause damage or accidents.

Do not use other therapeutic equipments whilst using this product. This may cause ill health or accidents.

Do not haul the chair itself or move with the casters on the floor where it easily gets scratches.

Do not overturn the chair or subject the chair to any strong shocks. Damage or injury may occur.

Do not operate where aerosol (spray) products are being used or where oxygen is being administered.

Use this appliance only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.

Do not carry this appliance by supply cord or use cord as a handle.

Never operate the appliance with the air openings blocked. Keep the air openings free of lint, hair, and the like.

Close supervision is necessary when this appliance is used by, on, or near children, invalids, or disabled persons.

WARNING

Never operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the appliance to a service center for examination and repair.

Keep the cord away from heated surfaces.

Use heated surfaces carefully. May cause serious burns. Do not use over insensitive skin areas or in the presence of poor circulation. The unattended use of heat by children or incapacitated persons may be dangerous.

Keep children away from extended foot support, back support (or other similar parts).

Connect this appliance to a properly grounded outlet only. See Grounding Instructions.

This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.

Children should be supervised to ensure that they do not play with the appliance.

Cleaning and user maintenance shall not be made by children without supervision

Please do not listen at high volume.

Listening at a high volume for a long time may damage your hearing.

IMPORTANT SAFETY INSTRUCTIONS

Grounding instructions

! DANGER

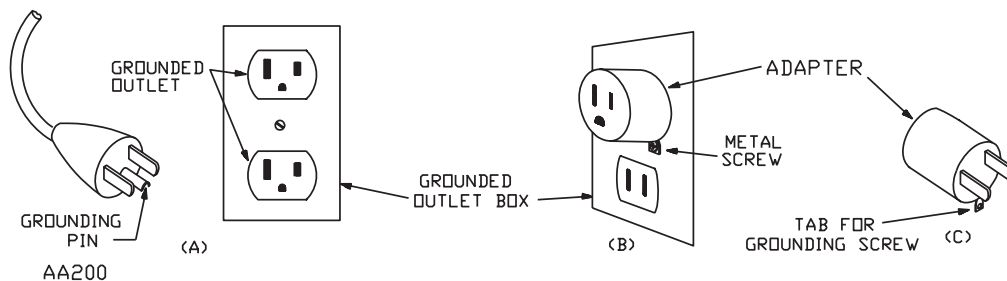
Improper connection of the equipment-grounding conductor can result in a risk of electric shock. Check with a qualified electrician or serviceman if you are in doubt as to whether the product is properly grounded. Do not modify the plug provided with the product – if it will not fit the outlet, have a proper outlet installed by a qualified electrician.

! WARNING

Ground should be installed firmly. When the ground is not installed, it may cause electric shock at the time of trouble and short circuit.

※ This product must be grounded. If it should malfunction or breakdown, grounding provides a path of least resistance for electric current to reduce the risk of electric shock. This product is equipped with a cord having an equipment-grounding conductor and a grounding plug. The plug must be plugged into an appropriate outlet that is properly installed and grounded in accordance with all local codes and ordinances.

Grounding methods

※ This product is for use on a nominal 120-V circuit, and has a grounding plug that looks like the plug illustrated in sketch A in Figure 86.1. A temporary adapter that looks like the adapter illustrated in sketches B and C may be used to connect this plug to a 2-pole receptacle as shown in sketch B if a properly grounded outlet is not available. The temporary adapter should be used only until a properly grounded outlet (sketch A) can be installed by a qualified electrician. The green colored rigid ear, lug, or the like extending from the adapter must be connected to a permanent ground such as a properly grounded outlet box cover. Whenever the adapter is used, it must be held in place by a metal screw.

When there is no ground terminal in the power outlet

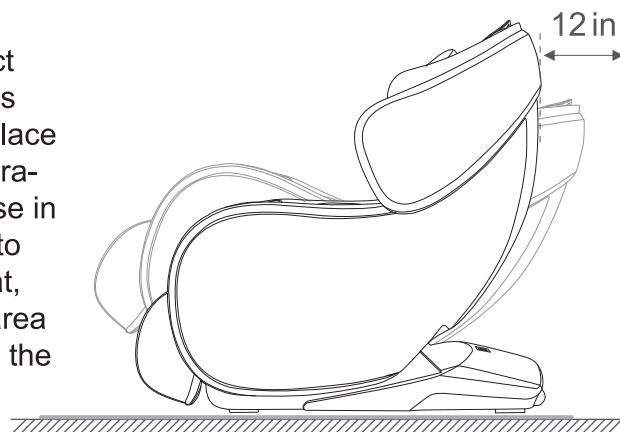
Before purchase, discuss with electric shop, dealer and do the grounding (D-Class<Class-D> installation work/cost).

Place massage chair

Installation location

Make sure there is enough space so that the massage chair can be tilted, and the back end of the massage chair should be at least 12in away from the wall.


Do not expose the massage chair to direct sunlight or high-temperature environments (such as in front of heating equipment). Place the massage chair away from high temperatures and direct sunlight. For long-term use in designated locations, it is recommended to lay soft cushions. When laying the mat flat, The size should be enough to cover the area where the machine contacts the floor and the area where the feet contact the floor.

Product lift instruction

Support the bottom sides of the stand, then lift and move. Since the machine is heavy, please be careful when carrying it to avoid straining your back. (This machine must be carried by two or more people)

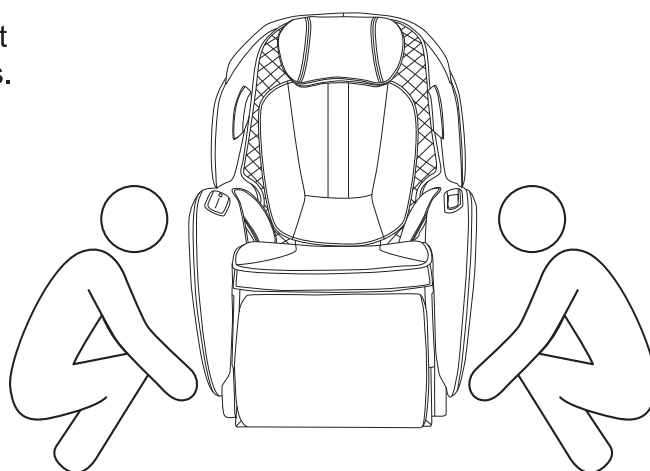1. Please lift the massage chair to move it on fragile surfaces such as wooden floors.
2. When moving on a wooden floor, it is recommended to lift. Avoid damage to the casters. When lifting and moving the floor to a designated position and lowering it, be careful to keep your hands out of the way to avoid damage and be careful of your feet being crushed. Do not let go of the machine until it is completely flat. The floor may be damaged. Therefore, it is recommended to place mats and other items on the floor.

Place massage chair

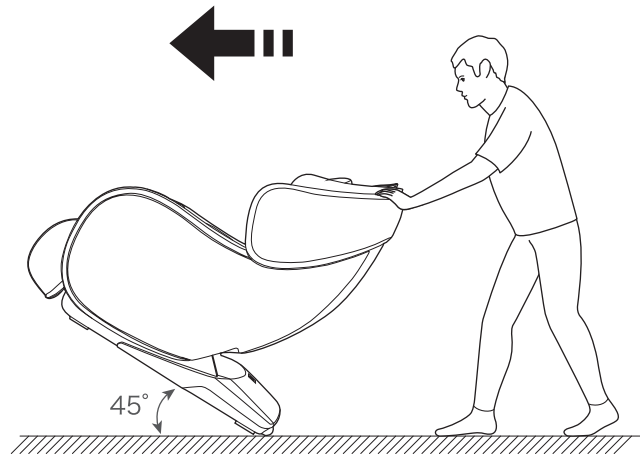
Mobile method

Use the rear casters to move the massage chair. According to the method shown in the picture, tilt the chair downwards and backwards about 45° to move it.

Note: Do not press down too much to avoid product damage due to complete tipping of the chair.

1. Before moving, you need to shut down the machine correctly, completely reset the backrest and leg mechanism, and then remove the power supply from the product.
2. The product must be left empty when moving. Do not allow people, pets or debris to stay on the product.
3. Use casters to move and do not cross obstacles with a height drop greater than 0.08in or gaps greater than 0.2in.
4. When using casters to move this product, a single continuous movement is not supported. If the distance exceeds 164ft, please move the product slowly and evenly.


Before use

Check around

Make sure there are no people, pets, or other objects around the unit.

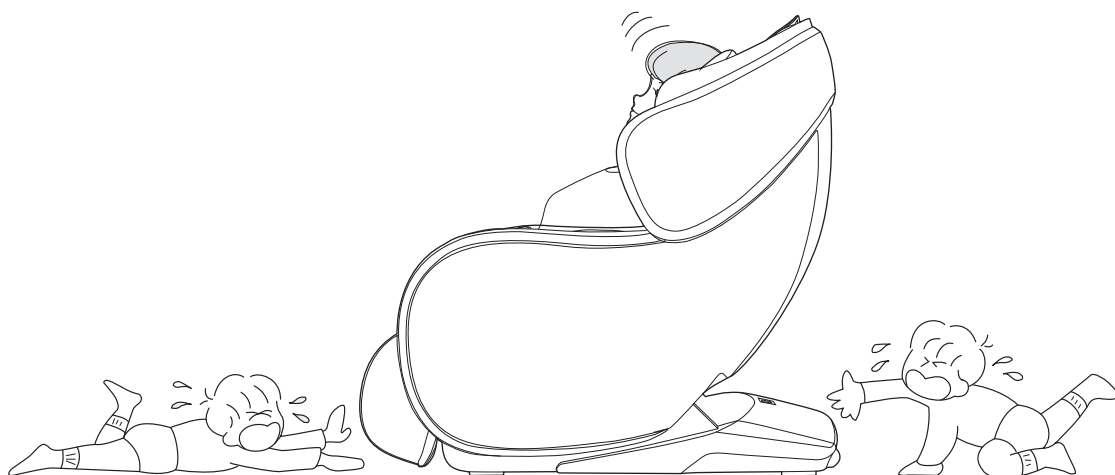CAUTION

Before use

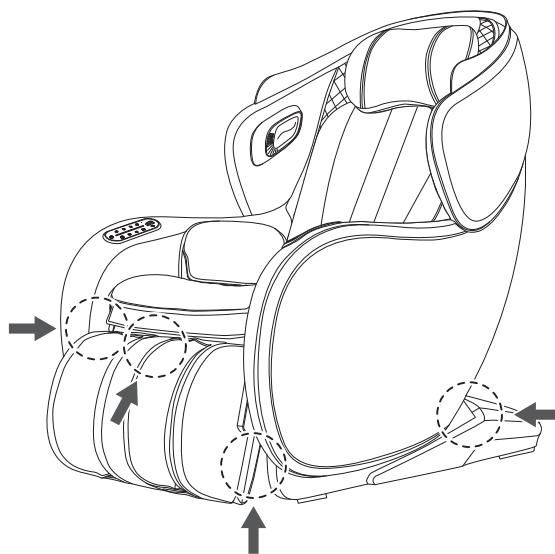
Anti-pinch safety1:warning signs

Please ensure there are no foreign objects in the gaps of the device before use.

Before starting the massage, make sure there are no obstructions in the calf, foot, and other areas. Under no circumstances should hands, feet, or head be inserted into the area between the calf mechanism and the body, as well as the area between the armrest and the space capsule.


Avoid allowing hands, feet, or head to become trapped behind the massage area fabric or leather covering.

Warning signs indicating the risk of pinching are affixed to the moving parts of the calf and the back cover of the chair.

CAUTION

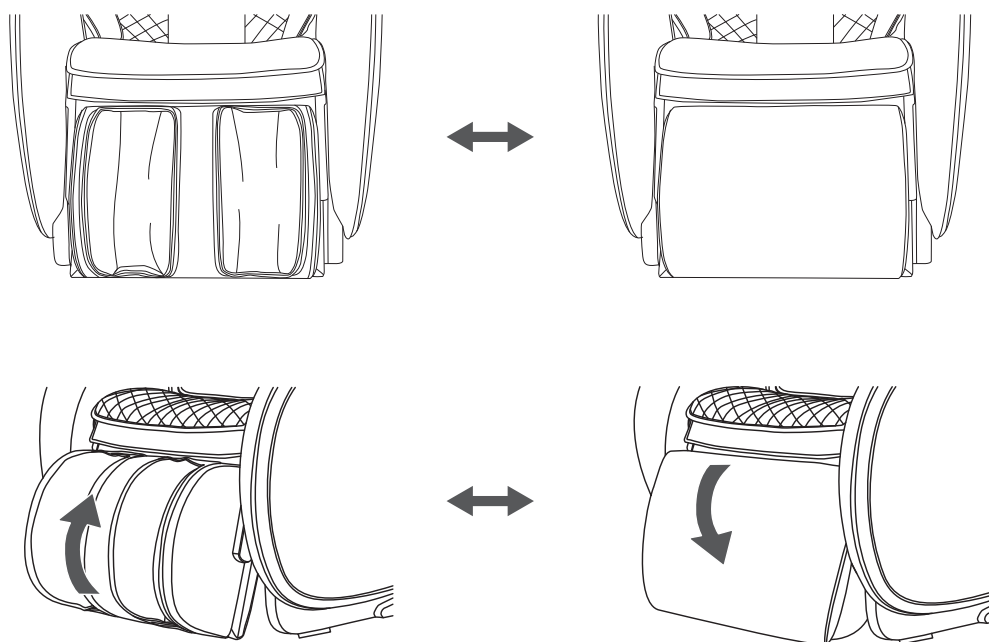Because the calf can flip, when flipped, Be careful to pinch your hands


Anti-Pinch Safety 2: Automatic Pinch Prevention Function

Pinch Sensing Explanation

The lower part of the backrest of this device is equipped with an anti-pinch function. During the reclining/upright movement, if an object touches the anti-pinch sensing area, the anti-pinch function is triggered, and the device enters pause mode. Remove the object from under the backrest, and press the pause button to resume the reclining/upright movement.

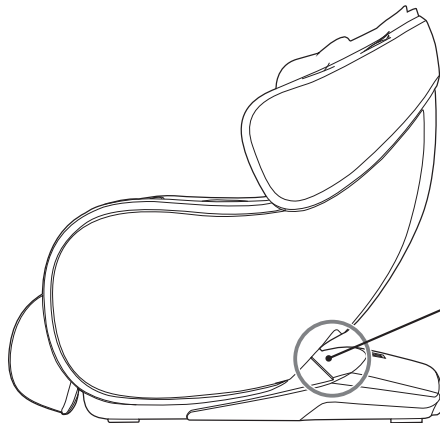CAUTION



Built-in anti-pinch sensor strip


Anti-pinch sensing position
Between the back cover and
the lower cover

Before use

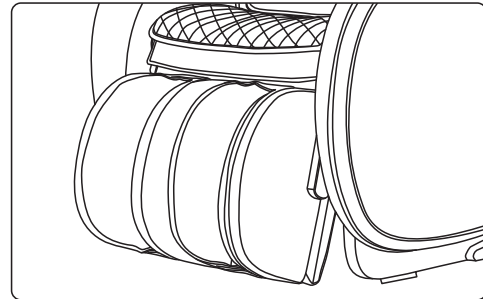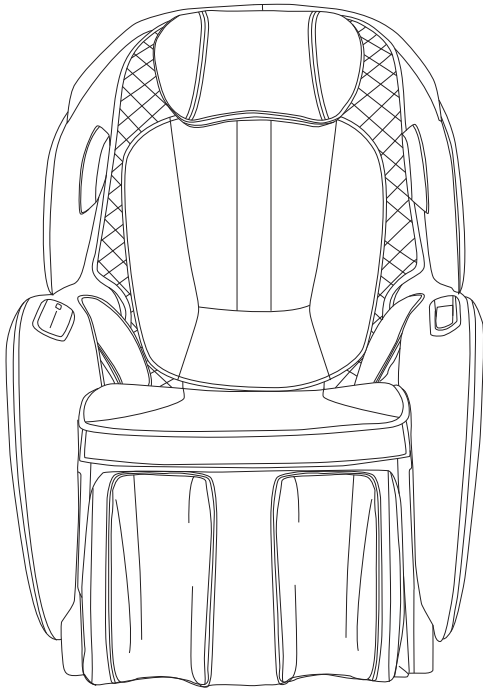Calf description

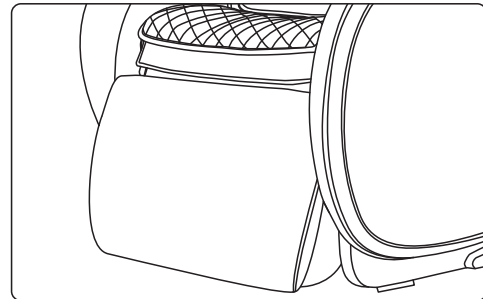Massage mode: Flip to enjoy calf air pressure massage


Leisure mode: flip it over and use it as a footrest for relax, easy storage, simple and convenient


Turn on the power

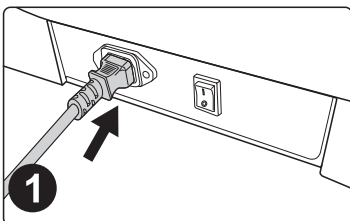
CAUTION

Product access power cord

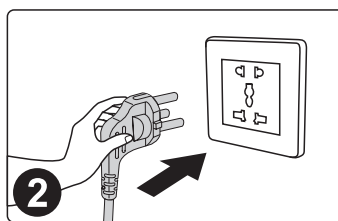Power on instructions

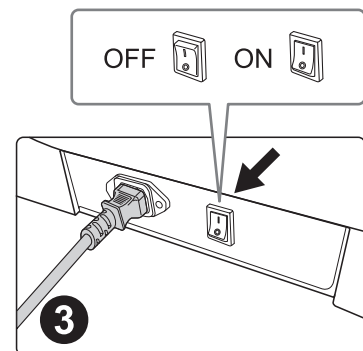Turn on the power switch ("I" position)

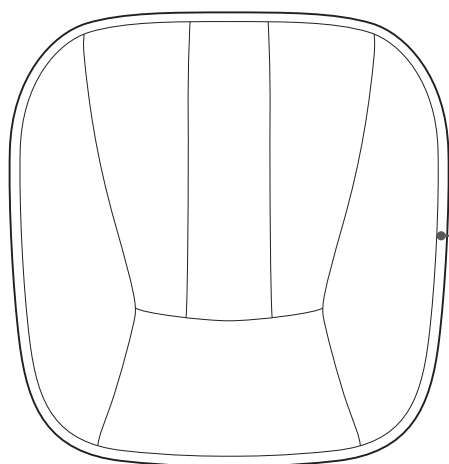
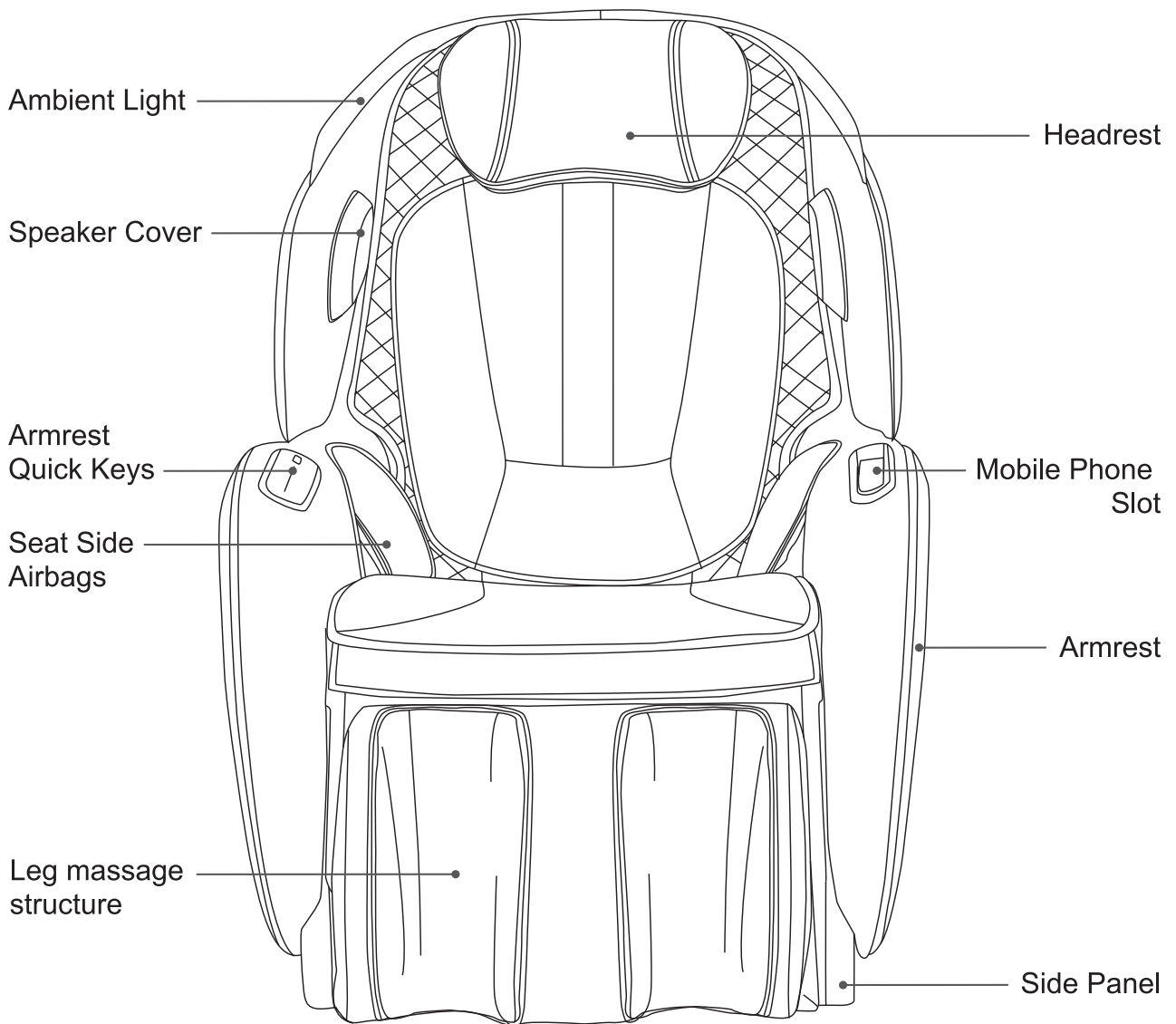
WARNING

- Please check whether the power cord and power plug are damaged before use.
- Please supply power to this product in accordance with the calibrated working power requirements of this product.
- It is strictly prohibited to use power supply other than the rated power supply of this product.
- It is strictly prohibited to connect various power conversion devices to power this product without authorization.

Product Structure


Component Descriptions:

Cushion
Can be placed on the back or on the seat.


Product Structure

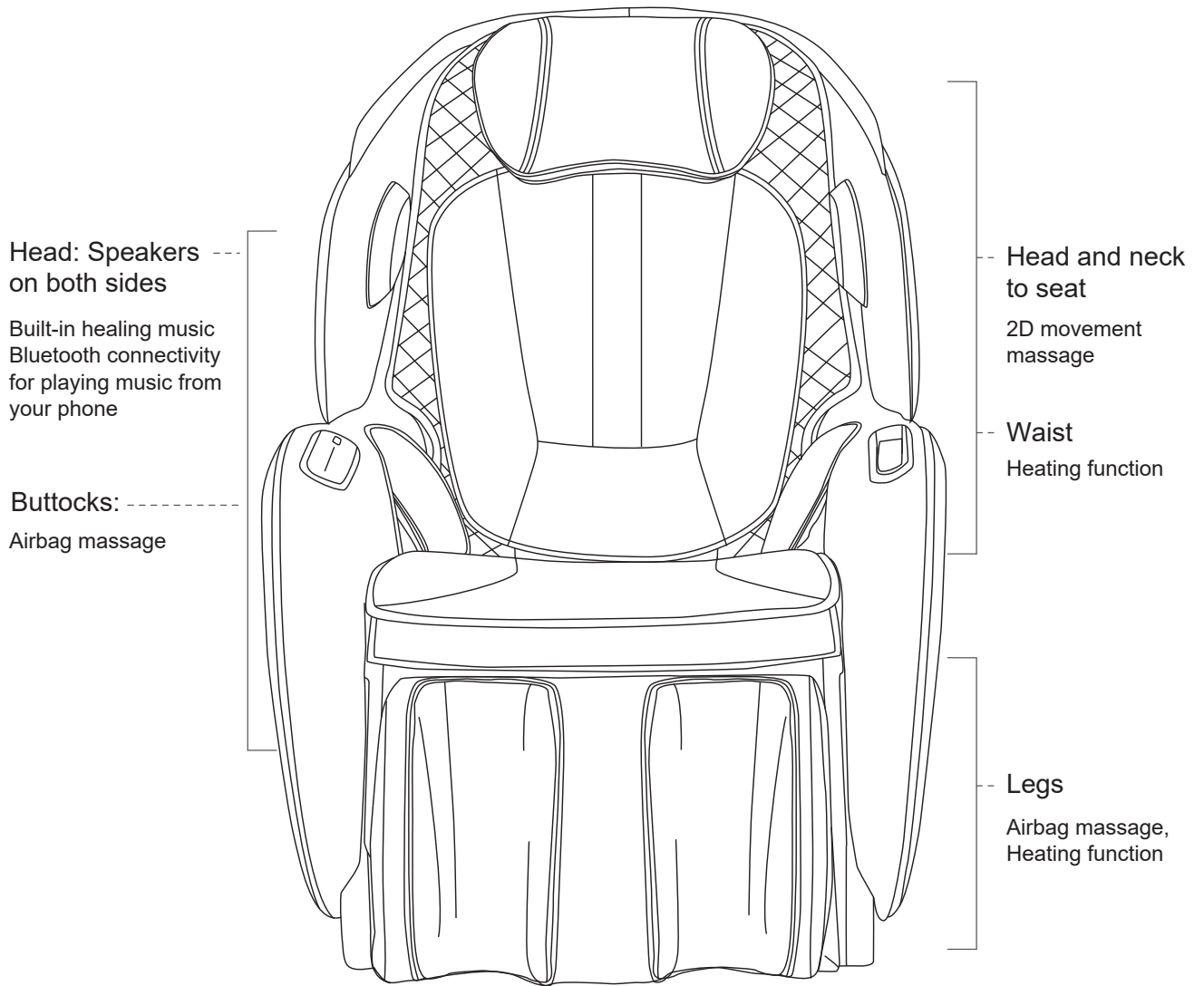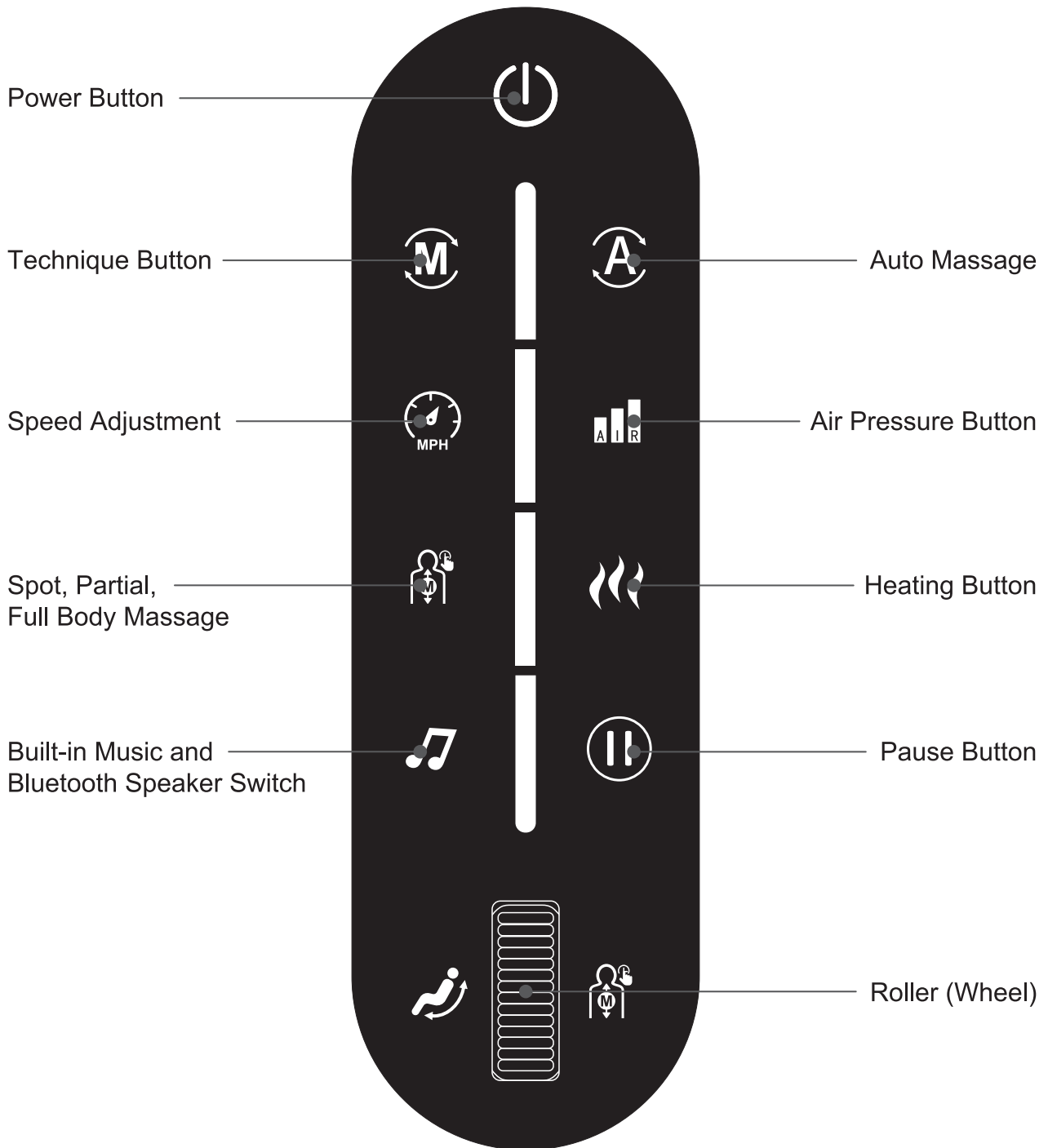Functional Distribution

Product Structure

I. Explanation of Right Armrest Quick Keys:

Product Structure

button	Button name	describe
	Power button	Short press to turn the device on or off. Long press to enter reclining mode.
	Technique Button	Manual massage program, refer to page 20 in the instruction manual for details.
	Speed Adjustment	Adjusts the massage speed during manual massage programs.
	Spot, Partial, Full Body Massage	During manual massage programs, press to switch massage areas. Short press to switch between full body and partial massage. Long press to adjust the position of the massage core.
	Built-in Music and Bluetooth Speaker Switch	Built-in healing music. Can also connect to a phone via Bluetooth for music playback.
	Auto Massage	Short press to switch automatic programs (see page 19 for details).
	Air Pressure Button	Turns air pressure on or off and switches between 4 intensity levels.
	Heating Button	Press to turn the heating function on or off.
	Pause Button	Press during a massage to urgently pause all massage actions. Press again to resume all massage actions.
	Roller (Wheel) Button	Function 1: Rotate the wheel forward or backward to control the rise or fall of the backrest. Function 2: During "Spot, Partial, Full Body Massage," rotate the wheel forward or backward to control the up and down position of the massage core.

Operation Instructions

II. Start Massage

1. Press the power button to activate the massage chair, and the backrest will automatically adjust its angle.

1.2 Select the auto program:

① Press the "A" key to choose the automatic program (you can select other automatic programs by pressing the "A" key again within 3 seconds).

② After selection, the chair will automatically conduct a shoulder position detection on the human body. Please maintain a sitting position that conforms to the massage chair.

③ Adjust the shoulder position. If the detected position deviates from the actual shoulder position, use the roller to adjust the position of the massage core until it aligns with the appropriate shoulder position. If there is no shoulder position adjustment within 10 seconds, the massage function will commence.

2.2 Select manual program:

Directly enter the manual program (without shoulder position detection action).

Adjusting Reclining Angle:

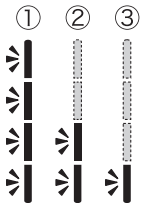
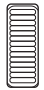
Function	Button	Describe
Rise		Rotate the wheel forward, and the backrest will slowly rise. Release the button to stop the ascent immediately. When approaching the limit, a short beep will sound as a prompt. If you continue to turn the wheel after reaching the limit, a short beep will indicate that you have reached the extreme position.
Lie Down		Rotate the wheel backward, and the backrest will slowly descend. Release the button to stop the descent immediately. When approaching the limit, a short beep will sound as a prompt. If you continue to turn the wheel after reaching the limit, a short beep will indicate that you have reached the extreme position.

Automatic massage:

Function	Button	Program Name	Describe
Automatic Massage	 Deep Blue	Full Body Fatigue Recovery	Slow and deep massage to eliminate fatigue and thoroughly relax.
	 Green	Whole Body Rhythmic	Rhythmic tapping motions to relax the entire body and release stress.
	 Violet	Whole Body Soothing	Primarily kneading motions for an overall soothing massage.
	 Orange	Full Body Stretching	Stretches muscles throughout the body, promoting flexibility.
	 Blue	Shoulder and Neck Massage	Focused massage on the shoulder and neck areas.
	 Yellow	Waist Massage	Focuses on massaging the waist area.
	 Purple	Pelvic Massage	Concentrates on massaging around the pelvic region.
	 Rose	Healing Relaxation	A relaxation program with a focus on airbag massage and stretching.

Operation Instructions

Manual Massage:

Function	Button	Describe
Technique	 Deep Blue	Kneading
	 Green	Variable-speed Kneading
	 Violet	Tapping
	 Orange	Variable-speed Tapping
	 Blue	Kneading + Tapping
	 Yellow	Variable speed kneading + variable speed Tapping
	 Purple	Circular Kneading
	 Rose	Shiatsu
Massage Range		Three adjustable modes (Full Body, Partial, Fixed Point).
		① Full-length reciprocating massage ② Small-range partial reciprocating massage ③ Fixed-point massage
		Adjustment of massage position (in fixed point and partial modes, the massage position can be adjusted).

Air Pressure Massage:

Function	Button	Describe
Air Pressure		Air pressure keys: 5Five adjustable modes, including off, and a cycle switch between four air pressure intensities.

Heating Massage:

Function	Button	Describe
Heating		Heating Key: Press this key to turn the heating function on or off. Warmth is felt after 3 minutes of activation.

Pause/Start Massage Key

Function	Button	Describe
Pause/Start		Press during a massage to pause all massage actions. Press again to resume all massage actions.

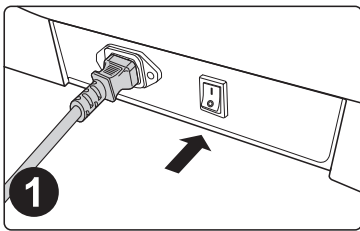
Note: When the speed key is pressed for 5 seconds, a "beep" sound is heard. Within 3 seconds, press the airbag key to turn the ambient light on/off.

III. Bluetooth Audio Playback Operation Instructions


1. Enter the main menu of the smart device, open "Settings," select the "Bluetooth" option, turn on "Bluetooth," and automatically "Search for new Bluetooth devices."
2. The device will search for the corresponding Bluetooth model (MR380-XXXXXXX). Click the pairing button. If asked to pair with the device, select "Pair" to complete the pairing.
3. Open the music player of the smart device and select a song to play.

IV. Ending the Massage


1. During the massage, press the armrest quick switch key to immediately turn off all massage functions, reset the backrest mechanism, or let the massage timer reach its limit, automatically turning off all massage functions and resetting the backrest mechanism.
2. Cut off the power supply to the entire machine (refer to the illustration for cutting off the entire machine power). Close the power switch (refer to the illustration). Unplug the power plug (refer to the illustration). Unplug the product connection power cord.

Turn off the power switch

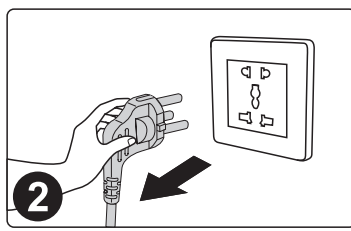Unplug the power plug
illustration

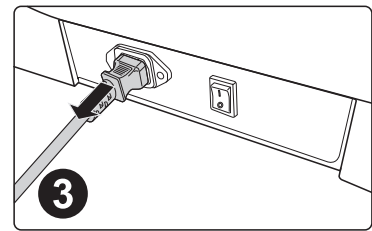Unplug the product from
the power cord

Cleaning and Maintenance

Clean PU leather and plastic parts

Wipe the skin with a soft dry cloth

■Do not use chemicals such as thinner, gasoline, etc. to clean the unit.

■When using commercially available leather care products (rags), please follow the instructions for use.

■If the PU leather is particularly dirty, wipe it in the following way:

① Immerse a soft cloth in water or hot water containing 3% to 5% neutral detergent, then thoroughly wring it out thoroughly.

② Use the above soft cloth to wipe the leather surface.

③ Wipe off the cleaning solution with a cloth soaked in clean water and thoroughly wrung soft cloth.

④ Wipe with a wrung soft cloth.

⑤ Let it air dry naturally.

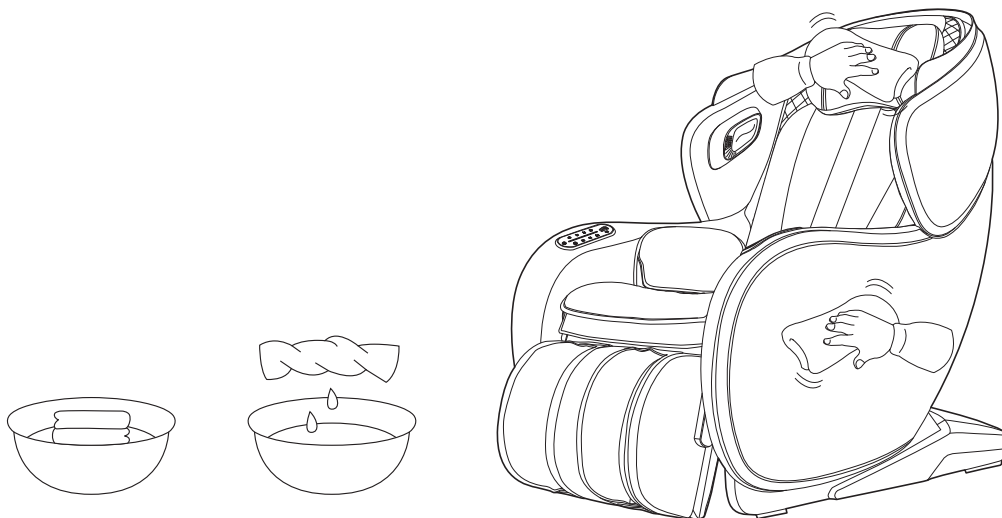
■When it is difficult to remove stains, please immerse commercially available melamine foam in neutral detergent, and then wipe the machine with it.

■Do not use a hair dryer to dry the surface.

■As the color of the fabric will stain the surface of the PU leather, be careful when wiping the chair with denim and colored fabrics.

■Do not keep these parts in contact with plastic for a long time, as this will cause discoloration.

■PU leather may be discolored, so when using hair dye, please cover the touched part with a towel.

Troubleshooting

Issue	The backrest reaches a certain position, pressing the corresponding adjustment button no longer changes it, and it emits a continuous "beep, beep" sound.
Solution	Due to the product's structural design and ergonomic requirements, there are restrictive protection measures for posture adjustment. A reminder sound will be emitted when reaching the limit.

Issue	After the machine is turned on, the functions are not running or some functions are not operating.
Solution	After pressing the machine's power button, it is necessary to manually select the automatic massage program or other operating states. If the manual operating mode is selected, functions that are not chosen may be in a closed state. Manually set the operating mode under manual mode.

Issue	The machine enters a pause state during massage.
Solution	The backrest of the machine has anti-pinch functionality, and it enters pause mode when foreign objects are detected. Remove the object from under the backrest and press the pause button to resume the massage.

Issue	The machine emits a squeaking friction sound during posture adjustment.
Solution	The entire product uses environmentally friendly and advanced anti-corrosive materials. Due to the inherent characteristics of these materials, slight friction noise may occur at relative motion points, which is considered normal. Note: If tearing sounds from fabric or material are heard during operation, immediately stop using it and seek professional assistance for inspection.

Issue	The massage rollers do not reach the position of the shoulders or neck.
Solution	If the head does not touch the headrest or the back does not touch the backrest, the shoulder position may be lower than the actual position during the body shape detection process. Sit in the deepest position of the chair, rest your head on the headrest, and then restart the operation from the beginning.

Issue	The machine is damaged. The power cord or power plug is abnormally heating.
Solution	To prevent accidents, please contact the authorized service center immediately.

For any uncertainties or questions regarding this product, feel free to contact the designated service personnel.

Product Specifications

Product name	CirC3
Model name	MR380
Power supply(50/60Hz)	AC110-240V
Rated time	20 min
Power consumption(50/60Hz)	100W
Safety structure	Class I
Maximum user weight	220lb
Country of origin	Made in China

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, maybe cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different form that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

 **Bluetooth**® The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by FUJIRYOKI is under license. Other trademarks and trade names are those of their respective owners.

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